DIASDAYGEL

SMOOTHING SKIN IRRITATIONS GEL

Instructions for Use

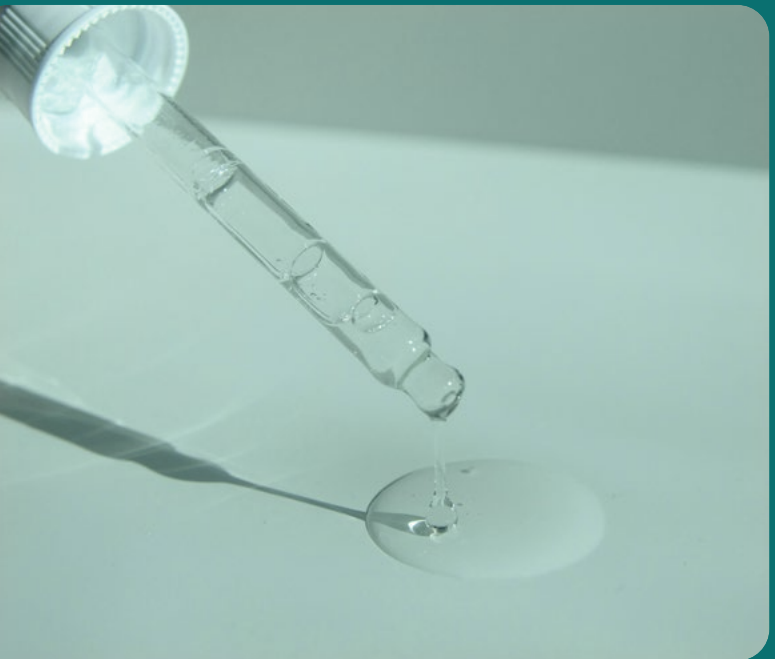
TABLE OF CONTENTS

SYMBOLS SECTION

SMOOTHING SKIN IRRITATIONS GEL

Instructions for Use

GLÄTTENDES HAUTIRRITATIONSGEL

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**Product description:**

DiasdayGel® Smoothing Skin Irritation is a silicone gel for treating scars (hypertrophic and keloid) and for preventing abnormal scar development after surgery or an accident.

Composition:

Dimethicone, Dimethicone Crosspolymer, Cyclopentasiloxane, Cyclohexasiloxane.

Properties:

- DiasdayGel® gel makes scars flatter, smoother and thinner and reduces complaints (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns).
- DiasdayGel® is a transparent gel which, thanks to its unique formulation, dries quickly and does not feel sticky.
- The gel provides a semi-occlusive layer on top of the skin, creating increased hydration of the skin (epidermis). The increased hydration ensures that the damaged scar skin recovers faster and reduces complaints.

Indications/areas of application:

DiasdayGel® Smoothing Skin Irritations is suitable for:

- the treatment and prevention of abnormal scars (hypertrophic and keloid).
- the treatment of burn scars.
- the treatment of surgical scars.
- the treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic objections).

Instructions for use:

- For optimum results preferably start DiasdayGel® treatment immediately after wound closure (after removing sutures and the disappearance of any scabs).
- A small amount of silicone gel per treatment is sufficient. For optimal therapy, apply a small amount of silicone gel to the scar twice a day with a smooth motion.
- Reapply DiasdayGel® after intensive activity (sports) and contact with water (showering, swimming).
- If desired, DiasdayGel® can be combined with make-up after drying.

How long should I use DiasdayGel® Smoothing Skin Irritation gel?

- It takes at least 3-6 months for a scar to normalize.
- Use DiasdayGel® as long as the scar is elevated, red or causes complaints.
- Consult with the practitioner/doctor about the duration of the therapy.

Duration of use:

- One vial of DiasdayGel® contains 15 ml.
- When used correctly, one vial contains enough for treatment lasting several weeks.
- The duration of use depends on the size of the scar to be treated.
- Multiple packs may therefore be necessary in the case of long-term therapy.

Special attention and precautions:

- Keep out of the reach of children.
- Avoid contact with eyes and mucous membranes.
- The silicone gel is only suitable for external use.
- Use the silicone gel only on closed skin (after the scab has disappeared) and never on open wounds.
- Do not use if the product is received with damaged packaging.

Side effects:

- In rare cases, the skin may react sensitively (skin irritation) to a silicone treatment. If this is the case, stop the treatment temporarily.
- If necessary, DiasdayGel® can be removed using a mild, skin-friendly soap.
- Start the treatment again once the irritation has disappeared.
- If skin problems persist consult your practitioner/doctor.

Storage:

- The expiry date is printed on the packaging.

Date of the last revision of instructions for use:

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DiasdayGel® Disclaimer of Warranty

DIASDAYGEL® MANUFACTURES ITS PRODUCTS (REFERRED TO AS THE "PRODUCTS") UNDER CONDITIONS THAT MEET APPLICABLE REGULATORY REQUIREMENTS, BUT HAS NO CONTROL OVER HOW THE PRODUCTS ARE ACTUALLY USED OR THE CONDITIONS UNDER WHICH THEY ARE USED. AS A RESULT, DIASDAYGEL® DISCLAIMS ALL PRODUCT WARRANTIES INCLUDING BUT NOT LIMITED TO ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. DIASDAYGEL® SHALL NOT BE LIABLE TO ANY PERSON OR ENTITY FOR ANY MEDICAL EXPENSES OR DAMAGES CAUSED BY ANY USE, DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER A CLAIM FOR SUCH DAMAGES IS BASED UPON WARRANTY, CONTRACT, TORT, OR OTHERWISE.

Limitation on Liability

IN NO EVENT WILL DIASDAYGEL®'S TOTAL LIABILITY TO PURCHASER OR ANY THIRD PARTY ARISING FROM THE SALE OF PRODUCTS EXCEED THE PURCHASE PRICE OF THE PRODUCTS SUBJECT TO THE CLAIM REGARDLESS OF THE THEORY OF LIABILITY OF THE CLAIM.

Limitation on Damages

DIASDAYGEL® IS NOT RESPONSIBLE TO PURCHASER OR ANY THIRD PARTY FOR ANY DIRECT, PUNITIVE, INCIDENTAL OR CONSEQUENTIAL DAMAGES BASED ON ANY CLAIMED BREACH OF WARRANTY, DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT.

Applicable law in your jurisdiction may limit the ability of a manufacturer to disclaim some warranties or exclude or limit damages. While the above limitations are an integral part of the business relationship between DiasdayGel® and the purchasers of its Products, nothing in the above is intended to contravene these provisions of applicable law. If any part or term of this Disclaimer of Warranty or the Limitations on Liability or Damages is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of all remaining provisions shall not be affected and shall be construed and enforced as if the Disclaimer of Warranty or the Limitations on Liability or Damages did not contain the particular part or term held to be invalid.

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